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Page 62

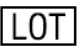
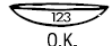
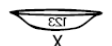
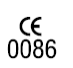
English

Important: Please read carefully and keep this information for future use. This Package Insert is intended for the Eye Care Professional, but should be made available to patients upon request. This insert contains important information about the correct usage of All ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® (galyfilcon A) and contains information about adverse reactions and contraindications.

Johnson & Johnson Medical Ltd provides ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® (galyfilcon A) and ACUVUE® ADVANCE® Brand Contact Lenses for ASTIGMATISM Patient Instruction Guides with further information for contact lens wearers. The Eye Care Professional should provide the patient with the Patient Instruction Guide that pertains to the patient's prescribed lens.

Symbols Key

The following symbols may appear on the labels or cartons of ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® and ACUVUE® ADVANCE® Brand Contact Lenses for ASTIGMATISM with HYDRACLEAR® Visibility Tinted with UV Blocker.

SYMBOL		DEFINITION	SYMBOL		DEFINITION
		See Instruction Leaflet			Fee Paid for Waste Management
		Use By Date (expiration date)			CAUTION: Federal law restricts this device to sale by or on the order of
		Batch Code			Manufactured by
		Sterile Using Steam or Dry Heat	ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR®		
DIA		Diameter			Visibility Tinted Lens Orientation Correct
BC		Base Curve			Visibility Tinted Lens inside Out
D		Diopter (lens power)	ACUVUE® ADVANCE® Brand Contact Lenses for ASTIGMATISM with HYDRACLEAR®		
		UV-Blocking	CYL		Cylinder Power
		Quality System Certification Symbol	AXIS		Axis
		Peel Back Foil			

Product Name	ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® and ACUVUE® ADVANCE® Brand Contact Lens for ASTIGMATISM with HYDRACLEAR®
Material	All ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® contain 53% polymer and 47% water.
Content	All ACUVUE® ADVANCE® Brand Contact Lens with HYDRACLEAR® are provided in sterile individual blister packages, with the lens immersed in buffered saline with methyl ether cellulose. DO NOT USE if the sterile blister package is opened or damaged.
Intended Use	<p>ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® and ACUVUE® ADVANCE® Brand Contact Lens for ASTIGMATISM are indicated for daily wear (less than 24 hours, while awake) for the correction of refractive ametropia (myopia and hyperopia) in aphakic or non-aphakic persons with non-diseased eyes.</p> <p>ACUVUE® ADVANCE® Brand Contact Lens with HYDRACLEAR® for eyes who may have 1 .00D of astigmatism or less.</p> <p>ACUVUE® ADVANCE® Brand Contact Lens for ASTIGMATISM for eyes that may have 0.50D to 2.50D of astigmatism.</p> <p>All ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® help protect against transmission of harmful UV radiation to the cornea and into the eye.</p>
Wear Schedule	<p>The Eye Care Professional should determine the appropriate wearing and replacement schedule based upon the response of the patient. We believe that more frequent replacement of our products achieves optimal health and performance.</p> <p>The recommended lens replacement for ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® and ACUVUE® ADVANCE® Brand Contact Lenses for ASTIGMATISM is every 2 weeks before disposal.</p> <p>All ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® are to be cleaned, rinsed and disinfected each time the lens is removed using a chemical disinfection system only.</p>

Contraindications

DO NOT prescribe ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® or ACUVUE® ADVANCE® Brand Contact Lenses for ASTIGMATISM when any of the following conditions of the eye exist. Inflammation, infection, irritation, any eye disease, injury or abnormality which affects the cornea, conjunctiva or eyelids, severe dry eyes, corneal hypoaesthesia, any systemic disease that may affect the eye, allergy that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions, or any use of medication that may interfere with contact lens wear.

Adverse Reactions

The patient should be informed that the following problems may occur when wearing contact lenses: eye may burn, sting or itch; reduced comfort after initial insertion; foreign body sensation; excessive watering of the eye; unusual secretion; redness of the eye; sensitivity to light; poor visual acuity; blurred vision, halos, rainbows around objects, dry eyes. There may be the potential for some temporary impairment due to peripheral infiltrates, peripheral corneal ulcers and corneal erosion. There may be the potential for other physiological observations, such as local or generalized oedema, corneal neovascularisation, corneal staining, injection, tarsal abnormalities, iritis and conjunctivitis, some of which are clinically acceptable in low amounts.

Warnings

(DW = less than 24 hours, while awake; EW = greater than 24 hours, including while asleep).

Proper use and care of contact lenses and lens care products, including lens cases, are essential for safe use. Problems from wearing contact lenses or using lens care products could result in serious injury to the eye. Please advise patients of the following about contact lens wear:

Patients should be advised of the following warnings pertaining to contact lens wear[†]:

- Problems with contact lenses or lens care products could result in serious injury to the eye. Patients should be cautioned that proper use and care of contact lenses and lens care products, including lens cases, are essential for the safe use of these products
- Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision.
- Studies have shown that the risk of ulcerative keratitis is greater for extended wear contact lens users than for daily wear users.
- When daily wear users wear their lenses overnight (outside the approved indication), the risk of ulcerative keratitis is greater than among those who do not wear them overnight.
- The overall risk of ulcerative keratitis may be reduced by carefully following directions for lens care, including cleaning the lens case.
- Studies have shown that the risk of ulcerative keratitis among contact lens users who smoke is greater than among non-smokers.

[†] *New England Journal of Medicine, September 21, 1989*

Before leaving the Eye Care Professional's office, the patient should be able to promptly remove lenses or should have someone else available who can remove the lenses for him or her. If eye discomfort, excessive tearing, vision changes, redness of the eye or other problems are experienced, contact lenses should be removed immediately and the wearer should promptly contact their Eye Care Professional. Contact lens wearers should see their Eye Care Professional routinely as directed.

Fluorescein, a yellow dye, should not be used while the lenses are (lens is) on the eye, as the lenses (lens) absorb this dye and become discoloured. Whenever fluorescein is used in eyes, the eyes should be flushed with a sterile saline solution that is recommended for in-eye use.

What to do if problems occur

PLEASE ADVISE YOUR PATIENT TO REMOVE CONTACT LENSES IMMEDIATELY AND SEEK ADVICE FROM THEIR EYE CARE PROFESSIONAL.

Important Patient Information

Precautions

Please inform your patients of the following:

Always wash your hands before insertion and removal of your contact lenses. Do not use lenses after expiry date 🕒. Do not use lenses if the sterile blister pack is opened or damaged; avoid spraying aerosols, e.g. hair spray, while wearing lenses; remove lenses when exposed to noxious or irritating vapour; cosmetic lotions, soap, creams or deodorants could cause irritation; regular follow-up visits with your Eye Care Professional are necessary to assure the continuing health of your eyes.

Ask your Eye Care Professional about wearing contact lenses during sporting activities, especially swimming and other water sports. Exposing contact lenses to water during swimming or while in a hot tub may increase the risk of eye infection from microorganisms. Also ask your Eye Care Professional about wearing contact lenses prior to using any medications in your eye. Always inform your employer of being a contact lens wearer. Some jobs may require use of eye protection equipment or may require that the patient not wear contact lenses. Certain medications, such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilizers and those for motion sickness, may cause dryness of the eye, increased lens awareness or blurred vision. Should such conditions exist, proper remedial measures should be prescribed. Depending on the severity, this could include the use of lubricating drops that are indicated for use with soft contact lenses or the temporary discontinuance of contact lens wear while such medication is being used. Oral contraceptive users could develop visual changes or changes in lens tolerance when using contact lenses.

All ACUVUE® ADVANCE® Brand Contact Lens with HYDRACLEAR® with UV Blocking are not a substitute for protective UV absorbing eyewear, such as UV absorbing sunglasses, because they do not completely cover the eye and the surrounding area.

Lens care directions

Lenses must be cleaned, rinsed and disinfected following removal before re-use. When any ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® are prescribed as frequent/planned replacement, the lens may be disinfected using a chemical disinfection system only (e.g. multi-purpose or hydrogen peroxide system). Since the lens material contains silicone, the wettability may differ when different lens care products are used. The Eye Care Professional should recommend a system that is appropriate for ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR®. Please review with the patient lens care directions, including basic information on lens care and lens case cleaning as well as specific instructions on the lens care regimen recommended for the patient.

Care for a sticking (non-moving) lens

If the lens sticks (stops moving), the patient should be instructed to apply a few drops of the recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after a few minutes, the patient should immediately consult the Eye Care Professional.

Care for a dried out (dehydrated) lens

If a frequent replacement lens is off the eye and exposed to air from 30 minutes to 1 hour or more, its surface will become dry and gradually become non-wetting. If this should occur, discard the lens and use a new one.

Emergencies

The patient should be informed that if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, the patient should:

FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THEIR EYE CARE PROFESSIONAL OR VISIT A HOSPITAL CASUALTY DEPARTMENT WITHOUT DELAY.

REPORTING OF ADVERSE REACTIONS

All serious adverse experiences and adverse reactions observed in patients wearing any ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® or experienced with the lenses should be reported to:

Johnson & Johnson Medical Limited
Pinewood Campus, Nine Mile Ride
Wokingham RG40 3EW
United Kingdom
Tel: 0800 022 4222 Fax: 0800 783 7029

Additional information

For additional information concerning all ACUVUE® ADVANCE® Brand Contact Lenses with HYDRACLEAR® and to order your contact lens Patient Instruction Guides free of charge, please contact customer service at the above address.

Manufactured for Johnson & Johnson Medical Limited by

Please refer to carton for site of manufacture

USA:

Johnson & Johnson Vision Care, Inc.
7500 Centurion Parkway
Jacksonville
Florida
32256
USA

IRELAND:

Johnson & Johnson Vision Care (Ireland)
The National Technology Park
Limerick
Ireland

EU Authorised representative

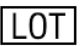
Johnson & Johnson Medical Limited
Pinewood Campus
Nine Mile Ride
Wokingham
RG40 3EW
United Kingdom

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©Johnson & Johnson Medical Limited. 2010
Fabricado para a Johnson & Johnson Medical Limited por