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© Bausch & Lomb Incorporated

PUREVISION®  **INSTRUCTIONS FOR USE**

This leaflet is intended to inform users of PureVision® hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

Product Name	Material	Water content	Packaging Solution
PureVision® (balafilcon A) Visibility Tinted Contact Lenses	Balafilcon A	36% by weight	Borate buffered saline
PureVision® Toric (balafilcon A) Visibility Tinted Contact Lenses	Balafilcon A	36% by weight	Borate buffered saline
PureVision® Multi-Focal (balafilcon A) Visibility Tinted Contact Lenses	Balafilcon A	36% by weight	Borate buffered saline
PureVision® 2 (balafilcon A) Visibility Tinted Contact Lenses	Balafilcon A	36% by weight	Borate buffered saline with 0.5% poloxamine

Description and wear application

These lenses are designed for daily wear with a replacement period as prescribed by the eye care professional. They may also be used as disposable lenses to be discarded after each removal or for extended wear for 1-30 days between removal, as recommended by the eye care professional. When reusing lenses they must be cleaned, rinsed and disinfected between removal and reinsertion.

The balafilcon A contact lenses can also be used for therapeutic use in the management of corneal epithelial disorders.

Always follow your eye care practitioner's recommended wear schedule and replacement periods.

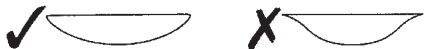
These lenses are supplied sterile and are intended by the manufacturer for reuse as prescribed by the eye care professional. They have been sterilized by steam.

If the packaging is damaged or opened DO NOT USE.

Insertion and removal instructions

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Pour the lens into the palm of your hand. Make sure the lens is the right way out.

Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens down to the white part of the eye and remove it. Use a rewetting drop if the lens is difficult to remove.

Caring for your lenses

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only soft contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes

- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner.

Warnings and precautions

Failure to follow your eye care practitioner’s recommended wear schedule and cleaning regime can give rise to greater risk of serious eye infections.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Dry eyes (inadequate tear fluid)
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes.

Contact your eye care practitioner before wearing contact lenses when:

- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication.

If you experience any of the following problems, remove your lenses **immediately**:

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light.

If any of these symptoms continue after removal, cleaning and reinsertion of the lens or insertion of a new lens, then remove the lens and **contact your eye care practitioner**. These symptoms can lead to more serious conditions if ignored.

Solution precautions

Do not use the Allergan Ultracare Disinfecting System or any of its components (Oxysept Ultracare Disinfecting Solution, Oxysept Neutralizing Tablets, and Ultrazyme Enzymatic Cleaner) to clean and disinfect Bausch + Lomb®. PureVision®2 (balafilcon A) Visibility Tinted Contact Lenses because the lens dimensions will be altered.

REFERENCE GUIDE FOR SYMBOLS USED ON LABEL AND CARTONS

Fee Paid for Waste Management per 94/62/EC

Federal (U.S.A) law restricts this device to sale by or on the order of a licensed professional.

Bausch & Lomb Incorporated
Rochester NY 14609, USA

Bausch & Lomb Incorporated
106 London Road
Kingston-upon-Thames KT2 6TN, UK

Date of Issue: 2010-09-29

0050

®/™ denote trademarks of Bausch & Lomb Incorporated
Other product/brand names are trademarks of their respective owners.

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