cleansing
disinfectant
preservative
protein-removing
rinsing'
wetting

the lens back in its case and fill to the indicated level with *PuraSoft*. Repeat the above steps for the right lens. Close the contact lens case and let the lenses soak for at least 4 hours or overnight.

· To completely eliminate any residual and make your lenses more comfortable, after disinfecting carefully rinse the lenses with *PuraSoft* before putting them in.

Precautions

· If the lenses remain in the same solution for more than 30 days repeat the steps in Directions.

· Once the bottle is opened it must be used within 2 months, after which throw any remaining solution away. Close the bottle after use and store at a temperature under 30° C. Do not use after the expiration date (see the bottom of the carton or the label) After use, rinse the contact lens case with *PuraSoft* and let it dry in the air.

· Always throw away any solution which remains in the case after removing your lenses. Replace your contact lens case at least every three months to reduce the risk of it becoming contaminated and thus also your lenses. Do not let the end of the dropper touch anything which could contaminate it. Make sure the seal on the bottle is intact, if it is not do not use the solution. The solution is sterile if the seal is intact.

Contraindications

There are no reported contraindications if the user does not have a specific allergy to one of the compounds.

Complications

In the event of irritations and/or problems remove your lenses, stop using the solution and contact your eye doctor.

Warnings

· Keep out of the reach of children. Before using eye drugs or medicines consult your eye doctor.

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MrLens GmbH
SOLITEC UK Ltd.