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Dinatrii edetas, detergentium, hydroxiethylcellulosum, alcohol isopropylicus 20%, conservans: E200 0.25%, excip. ad sol.

50 ml**Fabbricante:**

Laboratorium Dr. G. Bichsel AG
Bahnhofstrasse 5a
CH-3800 Interlaken

Distributore:

Contopharma AG
CH-3800 Interlaken
www.contopharma.ch

Information for Contact Lens Wearers

CONTOPHARMA «i-clean!» Cleaning Solution

This cleaning solution is suitable for the gentle cleaning of all current types of rigid gas-permeable as well as hydrophilic soft contact lenses. To date, there are no known lens types on which the solution should not be used.

Warnings:

To avoid complications with the system eyes/contact lenses, contact lens care products should be used only after consulting with a contact lens specialist, taking into consideration the lens material used. Wearing contact lenses that have not been cleaned properly or are damaged can lead to serious injury to the eyes or visual impairment. If a reddening of the eye occurs on the contact lens wearing surface or the connective tissue, one should consult an ophthalmologist or a contact lens specialist immediately.

Special Instructions for Use:

-  Avoid any contact of CONTOPHARMA «i-clean!» Cleaning Solution with your eyes.
- If the «i-clean!» cleaning solution should get into your eyes, flush your eyes with saline solution and consult your contact lens specialist and/or an ophthalmologist if any irritation persists.
- Once the bottle has been opened, the solution may not be used for more than 60 days (make a note of the first use on the bottle). Any remaining contents after 60 days should be properly disposed of.
- Do not use a bottle if the security seal has been damaged.
- Store the solution at room temperature (15°–25°C) and protect from direct light.
- Observe general hygiene measures such as hand washing before lens care.
- To prevent possible microbial contamination of the solution, do not touch the tip of the bottle. Close the bottle immediately after using and keep tightly closed.
- Keep out of the reach of children.
- The solution should not be used for storing contact lenses!

Instructions for Use:

Please observe the following instructions carefully:

1. Wash hands thoroughly and dry with a clean towel. Do not use enriched soaps for the hand washing.
2. Place several drops (ca. 2–4) of CONTOPHARMA «i-clean!» Cleaning Solution on both sides of the contact lens.
3. **Rigid gas-permeable contact lenses:**
Rub between the index finger, middle finger and the thumb for 20–30 seconds.
- Hydrophilic soft contact lenses:**
Clean both sides with the palm of one's hand with the aid of the index finger of the other hand for 20–30 seconds.
4. Thoroughly rinse off any dirt particles that have been loosened and the cleaning solution with a rinsing solution suitable for your lens type or a sterile saline solution.
 Rigid gas-permeable contact lenses can be used immediately after the thorough rinsing.
 Hydrophilic soft contact lenses should be stored in a suitable conditioning solution for 1–2 hours before insertion.
5. Close the bottle immediately after use.
6. Store the cleaned contact lenses in the proper solution in a contact lens storage case.
7. Before inserting the contact lenses, rinse once again with the proper rinse solution or with sterile saline solution.

To date no side-effects for the CONTOPHARMA «i-clean!» Cleaning Solution have been observed when used according to instructions.

Composition:

Dinatrii edetas, detergentium, hydroxiethylcellulosum, alcohol isopropylicus 20%, conservans: E200 0.25%, excip. ad sol.

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